RadiForce®

RX840 RX840-AR RX840-MG

Color LCD Monitor

English

Deutsch

Français

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Instructions for Use: ■ Setup Manual

Important: Please read this Setup Manual and the User's Manual stored (on the CD-ROM) carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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Compatible Resolutions

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The monitor supports the following resolutions.

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*3 Single Link

*4 Dual Link

A graphics board in conformance with the VESA standards is required.

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Color LCD Monitor

SAFETY SYMBOLS

This manual and this product use the safety symbols below. They denote critical information. Please read them carefully.

	WARNING Failure to abide by the information in a WARNING may result in serious injury and can be life threatening.		CAUTION Failure to abide by the information in a CAUTION may result in moderate injury and/or property or product damage.
	Indicates an attention to be required. For example, the symbol  illustrates the hazard type such as “the risk of electric shock”.		
	Indicates a prohibited action. For example, the symbol  illustrates a particular prohibited action such as “Do not disassemble”.		
	Indicates a mandatory action that must be followed. For example, the symbol  illustrates the notification of general prohibition such as “Grounding the unit”.		

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.


EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation’s receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

⚠ PRECAUTIONS

● IMPORTANT

- This product has been adjusted specifically for use in the region to which it was originally shipped. If the product is used outside the region, it may not operate as specified in the specifications.
- To personal safety and proper maintenance, please read carefully this section and the caution statements on the monitor.
- Please retain this document for future reference.

Location of the Caution Statements

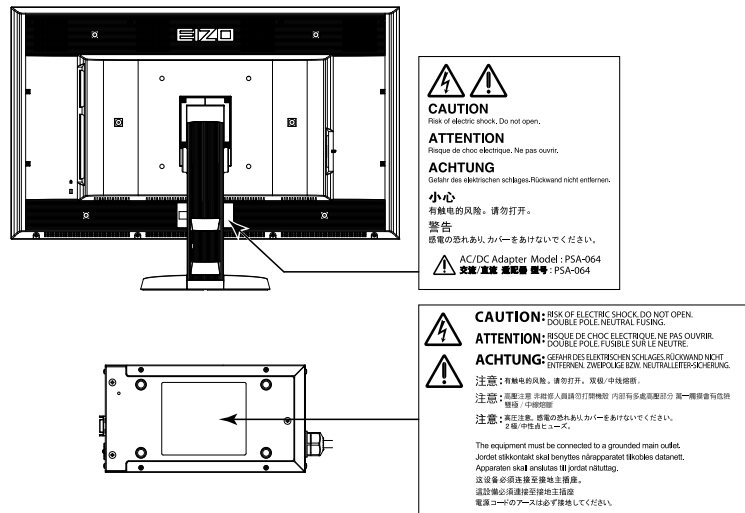⚠ WARNING

If the unit begins to emit smoke, smells like something is burning, or makes strange noises, disconnect all power connections immediately and contact your dealer for advice.

Attempting to use a malfunctioning unit may result in fire, electric shock, or equipment damage.

Do not open the cabinet or modify the unit.

Opening the cabinet or modifying the unit may result in fire, electric shock, or burn.

Do not turn the bushing for fixing the AC adapter power cable.

Not doing so may result in fire, electric shock, or equipment damage.

Refer all servicing to qualified service personnel.

Do not attempt to service this product yourself as opening or removing covers may result in fire, electric shock, or equipment damage.

Keep small objects or liquids away from the unit.

Small objects accidentally falling through the ventilation slots into the cabinet or spillage into the cabinet may result in fire, electric shock, or equipment damage. If an object or liquid falls/spills into the cabinet, unplug the unit immediately. Have the unit checked by a qualified service engineer before using it again.

Place the unit at the strong and stable place.

A unit placed on an inadequate surface may fall and result in injury or equipment damage. If the unit falls, disconnect the power immediately and ask your dealer for advice. Do not continue using a damaged unit. Using a damaged unit may result in fire or electric shock.

Use the unit in an appropriate location.

Not doing so may result in fire, electric shock, or equipment damage.

- Do not place outdoors.
- Do not place in the transportation system (ship, aircraft, trains, automobiles, etc.)
- Do not place in a dusty or humid environment.
- Do not place in a location where water is splashed on the screen (bathroom, kitchen, etc.).
- Do not place in a location where the steam comes directly on the screen.
- Do not place near heat generating devices or a humidifier.
- Do not place in a location where the product is subject to direct sunlight.
- Do not place in an inflammable gas environment.

**To avoid danger of suffocation, keep the plastic packing bags away from babies and children.****Use the enclosed AC adapter.**

The enclosed AC adapter (PSA-064) is for use with this product only. Do not use the AC adapter with other equipment. Connecting to power sources that do not match the power ratings of the AC adapter may result in fire or electric shock.

Use the enclosed power cord and connect to the standard power outlet of your country.

Be sure to remain within the rated voltage of the power cord. Not doing so may result in fire or electric shock.

Power supply: 100-240Vac 50/60Hz

To disconnect the power cord or adapter power cable, grasp the plug firmly and pull.

Tugging on the cord or cable may damage and result in fire or electric shock.

**The equipment must be connected to a grounded main outlet.**

Failure to do so may result in fire or electric shock.

**Use the correct voltage.**

- The unit is designed for use with a specific voltage only. Connection to another voltage than specified in this User's Manual may cause fire, electric shock, or equipment damage.
Power supply: 100-240Vac 50/60Hz
- Do not overload your power circuit, as this may result in fire or electric shock.

Handle the power cord and AC adapter with care.

Handle the power cord and AC adapter with care.

Do not place heavy objects on, pull or tie the power cord or the AC adapter. Use of a damaged cord or AC adapter may result in fire or electric shock.

**For the electrical safety, do not connect or disconnect the power cord in the presence of patients.****Never touch the plug, AC adapter or power cord during a thunderstorm.**

Touching them may result in electric shock.

**When attaching an arm stand, please refer to the user's manual of the arm stand and install the unit securely.**

Not doing so may cause the unit to become unattached, which may result in injury or equipment damage. Before installation, make sure that desks, walls, and others an arm stand is fixed on have adequate mechanical strength. When the unit is dropped, please ask your dealer for advice. Do not continue using a damaged unit. Using a damaged unit may result in fire or electric shock. When reattaching the tilt stand, please use the same screws and tighten them securely.

Handle the batteries for the remote control with care. (Only for the monitor with a remote control)

Not doing so may result in the battery explosion or the liquid leaking.

- Do not disassemble, heat, or throw the provided batteries in water.
- Install/change batteries in a correct way.
- When replacing batteries, replace all of them at the same time with new batteries of the same brand and type.
- Insert batteries with correct direction of the plus (+) and minus (-) sides.
- Do not use the battery with coating damaged.
- Dispose of your battery in a designated disposal area.

Do not touch a damaged LCD panel directly with bare hands.

The liquid crystal that may leak from the panel is poisonous if it enters the eyes or mouth. If any part of the skin or body comes in direct contact with the panel, please wash thoroughly. If some physical symptoms result, please consult your doctor.

Fluorescent backlight lamps contain mercury (the products that have LED backlight lamps contain no mercury), dispose according to local, state or federal laws.

Exposure to elemental mercury can result in effects on the nervous system, including tremor, memory loss, and headache.

CAUTION

Handle with care when carrying the unit.

Disconnect the power cord and cables when moving the unit. Moving the unit with the cord attached is dangerous. It may result in injury.

Carry or place the unit according to the correct specified methods.

- When carrying the unit, grasp and hold firmly as shown in the illustration below.
- Do not unpack or carry the unit only by a single person, since the large size unit is so heavy.

Dropping the unit may result in injury or equipment damage.

Do not block the ventilation slots on the cabinet.

- Do not place any objects on the ventilation slots.
- Do not install the unit in a closed space.
- Do not use the unit laid down or upside down.

Blocking the ventilation slots prevents proper airflow and may result in fire, electric shock, or equipment damage.

Do not touch the plug or AC adapter with wet hands.

Doing so may result in electrical shock.

Use an easily accessible power outlet.

This will ensure that you can disconnect the power quickly in case of a problem.

Mind that the AC adapter becomes hot during use.

- Do not cover or place anything on top of the AC adapter. Do not place the AC adapter on top of things that trap heat such as carpets, blankets, etc. Keep the AC adapter away from direct sunlight and heat sources such as heaters. Not doing so may result in fire.
 - Do not touch with bare hands. Doing so may result in burns.
 - Before moving the monitor, be sure to turn off the power switch, disconnect the power plug from the power outlet, and wait until it has cooled completely.
-

Do not suspend the AC adapter in midair.

Using it while suspended in midair may result in fire or electric shock.

Periodically clean the area around the power plug and the ventilation slot of the AC adapter.

Dust, water, or oil on the plug may result in fire.

Unplug the unit before cleaning it.

Cleaning the unit while it is plugged into a power outlet may result in electric shock.

If you plan to leave the unit unused for an extended period, disconnect the power cord from the wall socket after turning off the power switch for the safety and the power conservation.

This product is only suitable for a patient environment, but not for contact with a patient.


Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.


Note

• Please keep the packaging box and materials for future movement or transport of the monitor.


- Monitor
- Power cord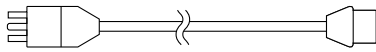


- Digital signal cable: DisplayPort - DisplayPort (PP300) × 2 pcs.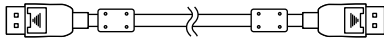

- Digital signal cable: DVI-D - DVI-D (Dual Link) (DD300DL) × 2 pcs.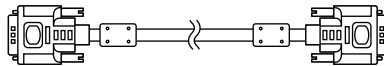


- USB cable : UU300

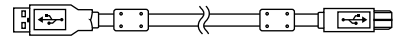- AC adapter (PSA-064)
- EIZO LCD Utility Disk (CD-ROM)
- Setup Manual (this manual)
- VESA Mounting Screws: M6 × 16: 4 pcs.
- Power cord holder
- Recycling Information

Controls and Functions


Monitor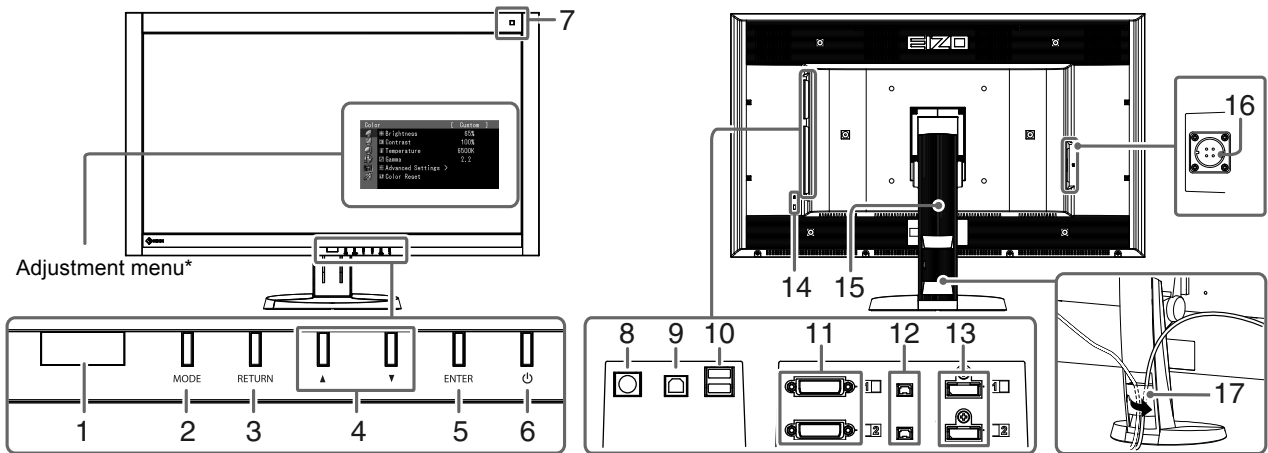

1. Presence Sensor	Detects the movement of a person in front of the monitor.
2. MODE button	Switches the CAL Switch mode.
3. RETURN button	Cancels the setting/adjustment and exits the Adjustment menu.
4. ▲▼ button	Chooses and increases/decreases adjustment settings for advanced adjustment using the Adjustment menu.
5. ENTER button	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
6. ⏻ button	<ul style="list-style-type: none"> • Turns the power on or off. • Indicates the monitor's operation status. Green: Operating, Orange: Power saving mode, Off: AC adapter main power/ power off
7. Ambient light sensor	Measures the ambient illuminance.
8. PS/2 connector	Used for maintenance. Do not use for other applications. This may cause the monitor to malfunction.
9. USB port (Up)	Connects the USB cable to use the software that needs USB connection, or to use USB Hub function.
10. USB port (Down)	Connects a peripheral USB device.
11. Input signal connector	DVI-D connector
12. DC5V Output	Supplies electrical power to the external devices.
13. Input signal connector	DisplayPort connector
14. Security lock slot	Complies with Kensington's MicroSaver security system.
15. Stand	Used to adjust the height and angle of the monitor screen.
16. Power connector	Used to connect the AC adapter power cable.
17. Cable holder	Covers the monitor cables.

* For instructions for use, refer to the User's Manual (on the CD-ROM).

AC adapter (PSA-064)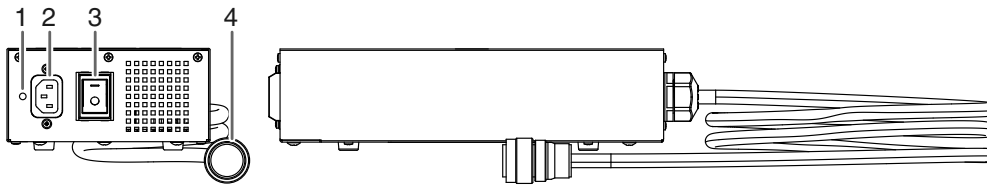

1. Power cord holder attachment hole	Used to attach the power cord holder.
2. Power connector	Connects the power cord.
3. Main power switch	Turns the main power on or off.
4. Power cable	Connects with the power connector on the monitor.

Connecting Cables

Attention

- Check that the monitor and the PC are powered off.
- When replacing the current monitor with this monitor, be sure to change the PC settings for resolution and vertical scan frequency to those that are available for this monitor. Refer to the compatible resolution table (back of cover page) before connecting the PC.


Note

- When connecting multiple PCs to this product, refer to the User's Manual (on the CD-ROM).

1. Connect the signal cables to the input signal connectors and PC.

Check the shape of the connectors, and connect the cables.

After connecting the signal cable, tighten the screws of the connectors to secure the coupling.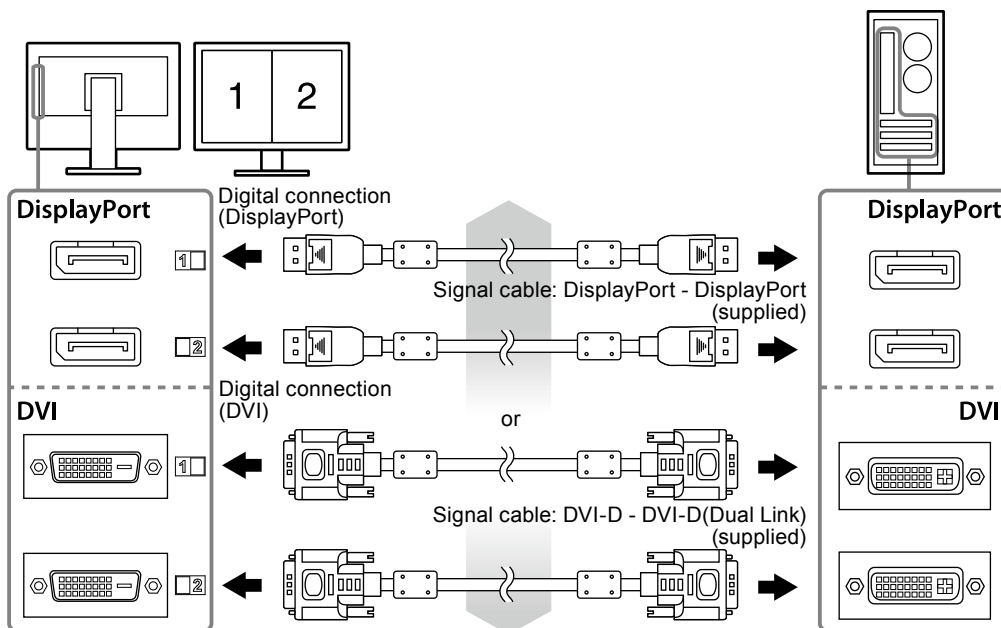

2. Connect the AC adapter power cable to the power connector on the monitor.


Attention

- Do not remove or connect the cable while the AC adapter main power is on.
- Do not install the AC adapter in the wrong orientation. The side with the EIZO logo should face up.

Check the shape of the monitor connector, and connect the power cable.

Tighten the screw for fixing the power cable to secure the power cable.

Connecting to the Monitor

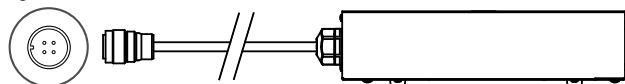3. Plug the power cord into a power outlet and the power connector on the AC adapter.

4. Turn on the AC adapter main power.

-: On, ○: Off

5. Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.

Connecting to the Monitor

Connecting to a Computer

6. Press to turn on the monitor.

The monitor's power indicator lights up green.

7. Turn on the PC.

The screen image appears.


If an image does not appear, refer to "No picture" (page 10) for additional advice.

Attention

- Turn off the monitor and PC after using them.
- For the maximum power saving, it is recommended that the Power button be turned off. Turning off the AC adapter main power switch or unplugging the power cord completely shuts off power supply to the monitor.

Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.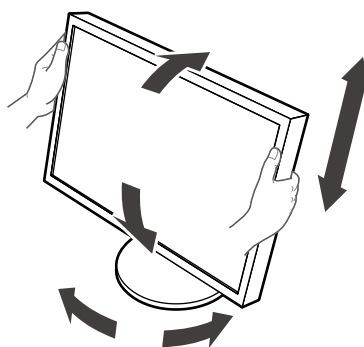

Attention

- Be sure that the cables are correctly connected.


Attaching a Power Cord Holder

Attach the power cord holder to prevent the power cord from falling off.


Attention

- Be sure to attach the power cord holder when the main power to the AC adapter is off or before the power cable is plugged into the power outlet.


- 1. Attach the supplied power cord holder to the power cord attachment hole on the AC adapter.**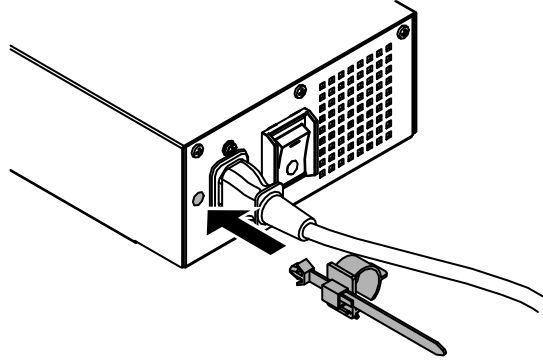

- 2. Adjust the position to secure the power cord holder referring to the figure shown below.**

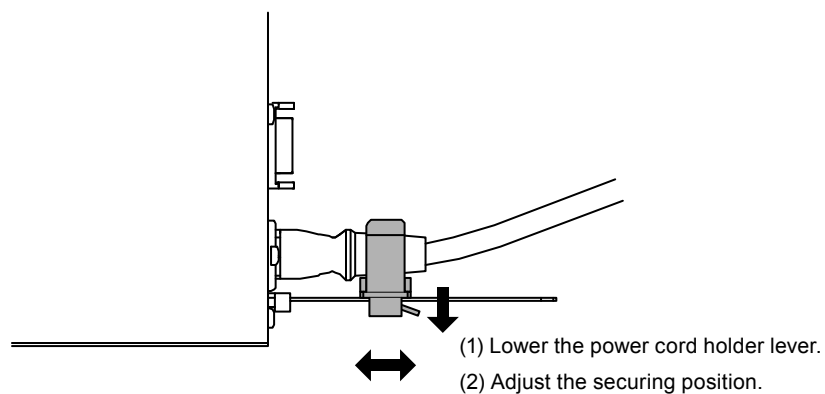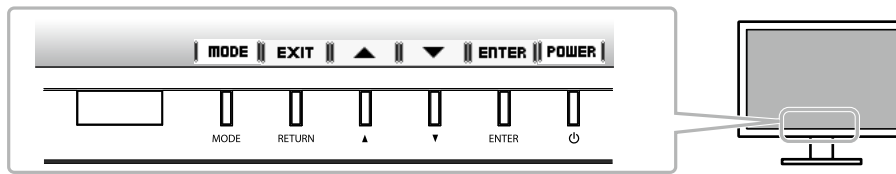
- 3. Secure the power cord holder to prevent the power cord from falling off.**

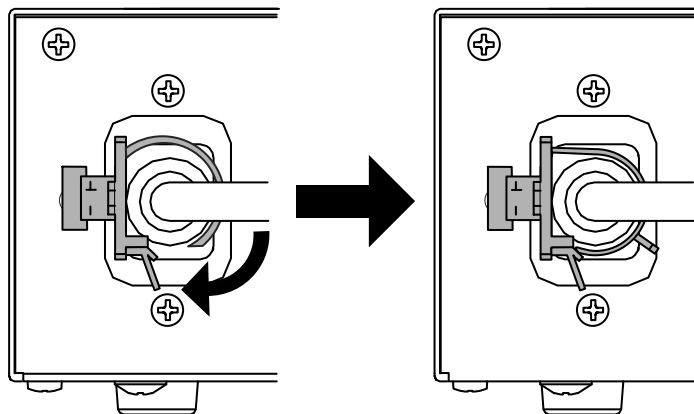*Check that the power cord holder is secured.

Showing Button Guide

Press the front buttons (except  button), the button guide appears above the button.


Note

- The button guide will continue to appear while the Adjustment menu or Mode menu is showing.
- The button guide is displayed differently depending on the selected menu or status.

Selecting Display Mode

This function allows easy selection of a desired mode according to monitor application.

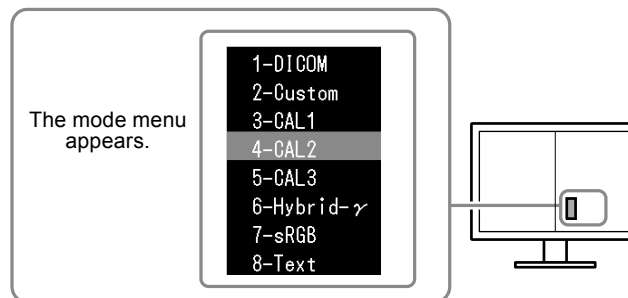Mode	Purpose
1-DICOM	Available to display in DICOM mode.
2-Custom	Select this to define a desired set of parameter settings.
3-CAL1/4-CAL2/5-CAL3	Displays the screen adjusted by calibration software.
6-Hybrid- γ	Automatically identifies the display area for the monochrome and color images on the same screen, and displays each set status.
7-sRGB	Suitable for color matching with sRGB compatible peripherals.
8-Text	Suitable for displaying texts for word processing or spreadsheets.

Procedure

1. Press MODE.
The mode menu appears at the bottom left of the right screen.
2. The mode among the list is highlighted in turn each time MODE is pressed.
You can switch the mode with  or  while the mode menu is displayed.

← 1-DICOM ↔ 2-Custom ↔ 3-CAL1 ↔ 4-CAL2 ↔ 5-CAL3 ↔ 6-Hybrid- γ ↔ 7-sRGB ↔ 8-Text ←

Controlling Monitor Quality

The quality control software “RadiCS LE” (for Windows) is included with this monitor.

To calibrate the monitor and manage the calibration history, install RadiCS LE with reference to the RadiCS LE User's Manual (on the CD-ROM). Use the RadiCS SelfQC function and built-in Integrated Front Sensor to perform periodic calibration and Grayscale Check for the monitor independently. For details, refer to the User's Manual (on the CD-ROM).

Attention

- Connect the monitor to the PC with a USB cable when using RadiCS LE.

Note

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit “RadiCS UX1”.

No picture

If a problem still remains after applying the suggested remedies, contact your local dealer.

Problem	Possible cause and remedy
<p>1. No picture</p> <ul style="list-style-type: none"> Power indicator does not light. <hr/> <ul style="list-style-type: none"> Power indicator is lighting green. <hr/> <ul style="list-style-type: none"> Power indicator is lighting orange. <hr/> <ul style="list-style-type: none"> Power indicator flashes orange and green. 	<ul style="list-style-type: none"> Check whether the power cord and the adapter power cable are connected properly. Turn the AC adapter main power switch on. Press . Turn off the AC adapter main power, and then turn it on again a few minutes later. Increase "Brightness", "Contrast", or "Gain" in the Adjustment menu. Switch the input signal. For details, refer to the User's Manual (on the CD-ROM). Operate the mouse or keyboard. Check whether the PC is turned on. When the Presence Sensor is set at a value from "Level-1" to "Level-4", the monitor may have shifted to the power saving mode. Come near to the monitor. There is a problem in the device that uses DisplayPort connection. Solve the problem, turn off the monitor, and then turn it on again. Refer to the user's manual of the device that uses DisplayPort connection for details.
<p>2. The message below appears.</p> <ul style="list-style-type: none"> This message appears when no signal is input. Example: <div data-bbox="316 1043 627 1384" data-label="Image"> </div>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <ul style="list-style-type: none"> The message shown left may appear, because some PCs do not output the signal soon after power-on. Check whether the PC is turned on. Check whether the signal cable is connected properly. Switch the input signal. For details, refer to the User's Manual (on the CD-ROM).
<ul style="list-style-type: none"> The message shows that the input signal is out of the specified frequency range. (Such signal frequency is displayed in magenta.) Example: <div data-bbox="316 1527 627 1917" data-label="Image"> </div> <p>fD: Dot clock fH: Horizontal scan frequency fV: Vertical scan frequency</p>	<ul style="list-style-type: none"> Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions"). Reboot the PC. Select an appropriate setting using the graphics board's utility. Refer to the manual of the graphics board for details.

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Symbols on the unit

Symbol	This symbol indicates	
	AC adapter main power switch:	Press to turn the monitor's main power off.
	AC adapter main power switch:	Press to turn the monitor's main power on.
	Power button:	Press to turn the monitor's power on or off.
	Alternating current	
	Alerting electrical hazard	
	CAUTION:	Refer to SAFETY SYMBOLS section in PRECAUTIONS.

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<http://www.eizo.com>

Copyright © 2011-2014 EIZO Corporation All rights reserved.

6th Edition-January, 2014 Printed in Japan.

00NOL690F1
(U.M-SUMRX840-4)

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EIZO GmbH