RadiForce® RX440

Color LCD Monitor

Instructions for Use: ■ Setup Manual

Important: Please read PRECAUTIONS, this Setup Manual and the User's Manual (on the CD-ROM) carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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Compatible Resolutions

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The monitor supports the following resolutions.

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*1 For details, refer to the User's Manual (on the CD-ROM). The applicable signal for the sub screen (Sub) for "PinP" differs.

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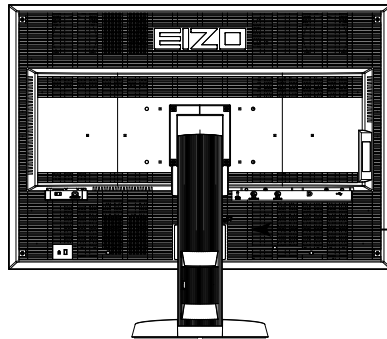
RadiForce® RX440

Color LCD Monitor

Location of the Caution Statements

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Symbols on the unit

Symbol	This symbol indicates
	Main Power Switch: Press to turn the monitor's main power off.
	Main Power Switch: Press to turn the monitor's main power on.
	Power button: Press to turn the monitor's power on or off.
	Alternating current
	Alerting electrical hazard
	CAUTION: Refer to SAFETY SYMBOLS section in PRECAUTIONS.
	WEEE marking: Product must be disposed of separately; materials may be recycled.
	CE marking: EU conformity mark in accordance with the provisions of Council Directive 93/42/EEC and 2011/65EU.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.

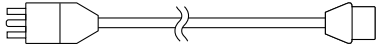
EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

Package Contents


Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local EIZO representative.

Note

- Please keep the packaging box and materials for future movement or transport of the monitor.
 - Monitor
 - Power cord
 - Digital signal cable : DisplayPort - DisplayPort (PP300)
 - Digital signal cable : DVI-D - DVI-D (Dual Link) (DD300DL)
 - USB cable : UU300
 - EIZO LCD Utility Disk (CD-ROM)
 - Setup Manual (this manual)
 - PRECAUTIONS
 - VESA Mounting Screws: M4 × 12: 2 pcs.

Controls and Functions

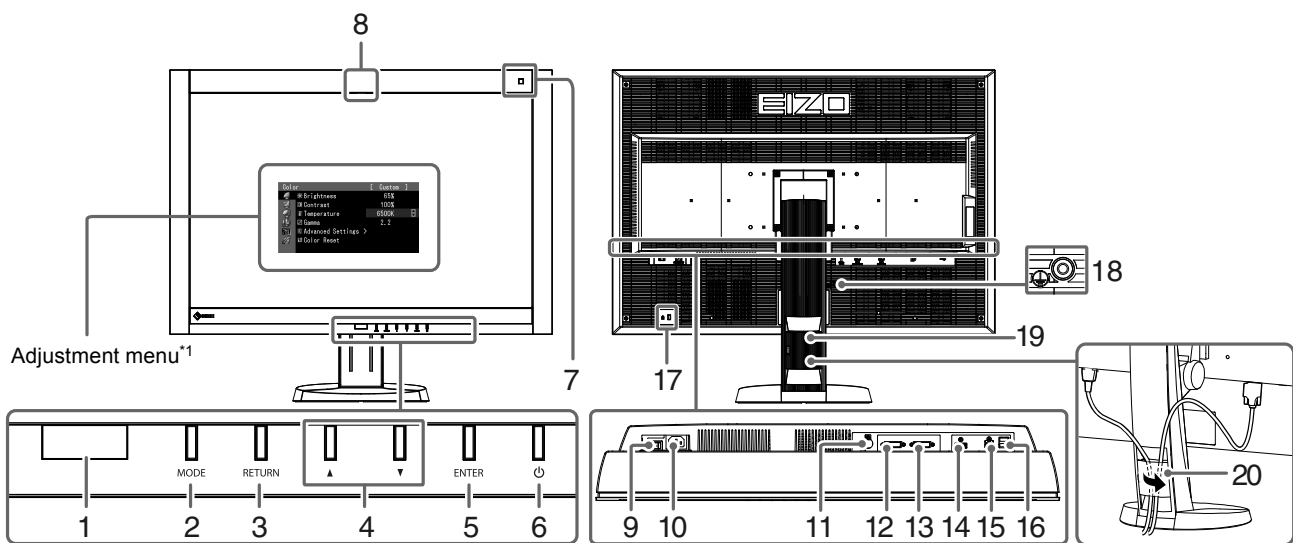1. Presence Sensor	Detects the movement of a person in front of the monitor.
2. MODE button	Switches the CAL Switch mode.
3. RETURN button	Cancels the setting/adjustment and exits the Adjustment menu.
4. ▲▼ button	Chooses and increases/decreases adjustment settings for advanced adjustment using the Adjustment menu.
5. ENTER button	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
6. ⏻ button	<ul style="list-style-type: none"> • Turns the power on or off. • Indicates the monitor's operation status. Green: Operating, Orange: Power saving mode, Off: Main power/power off
7. Ambient Light Sensor	Measures the environmental illuminance.
8. Integrated Front Sensor (slide type)	Used for calibration and Grayscale Check.
9. Main power switch	Turns the main power on or off.
10. Power connector	Connects the power cord.
11. PS/2 connector	Used for maintenance. Do not use for other applications. This may cause the monitor to malfunction.
12. Input signal connector	DVI-D connector (DVI-2 : Single Link)
13. Input signal connector	DVI-D connector (DVI-1 : Dual Link)
14. Input signal connector	DisplayPort connector
15. USB upstream port	Connects the USB cable to use the software that needs USB connection, or to use USB Hub function.
16. USB downstream port	Connects a peripheral USB device.
17. Security lock slot	Complies with Kensington's MicroSaver security system.
18. ⚡ mark	Protective earth terminal
19. Stand	Used to adjust the height and angle of the monitor screen.
20. Cable holder	Covers the monitor cables.

*1 For instructions for use, refer to the User's Manual (on the CD-ROM).

Connecting Cables

Attention

- Check that the monitor and the PC are powered off.
- When replacing the current monitor with this monitor, be sure to change the PC settings for resolution and vertical scan frequency to those that are available for this monitor. Refer to the compatible resolution table (back of cover page) before connecting the PC.

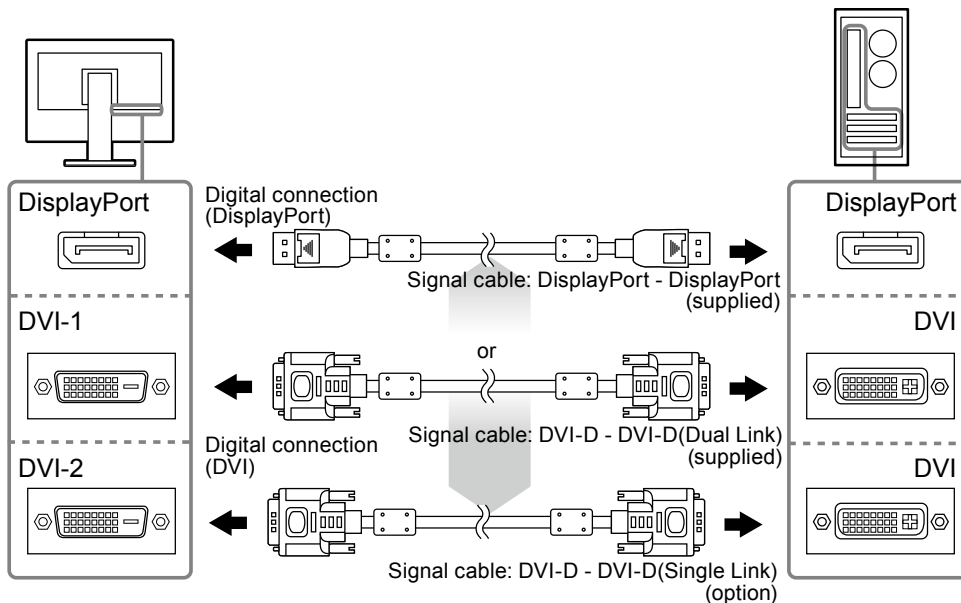
Note

- When connecting multiple PCs to this product, or displaying the images from two signals at the same time using one PC, refer to the User's Manual (on the CD-ROM).
- When two signals are displayed at the same time using the "PinP" or "PbyP" functions, the following input signals are displayed on the "Main" and "Sub" screens. For details, refer to the User's Manual (on the CD-ROM).
 - Main: DisplayPort or DVI-1
 - Sub: DVI-2

1. Connect the signal cables to the input signal connectors and PC.

Check the shape of the connectors, and connect the cables.

After connecting the signal cable, tighten the screws of the connectors to secure the coupling.

2. Plug the power cord into a power outlet and the power connector on the monitor.

3. Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.

4. Press to turn on the monitor.

The monitor's power indicator lights up green.

5. Turn on the PC.

The screen image appears.

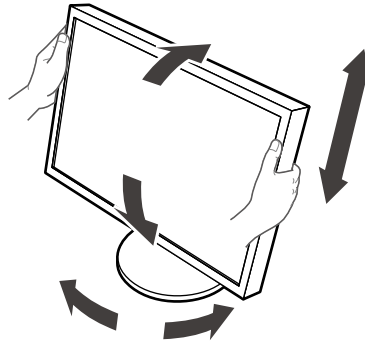
If an image does not appear, refer to "No picture" (page 6) for additional advice.

Attention

- Turn off the monitor and PC after using them.
 - For the maximum power saving, it is recommended that the Power button be turned off. Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.
-

Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.


Attention

- Be sure that the cables are correctly connected.
-

Showing Button Guide

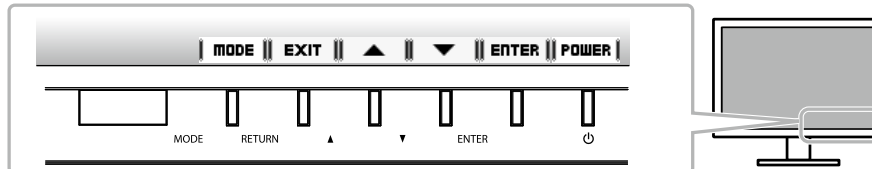Press the front buttons (except  button), the Button Guide appears above the button.

Note

- The Button Guide will continue to appear while the Adjustment menu or mode menu is showing.
 - The Button Guide is displayed differently depending on the selected menu or status.
-


Selecting Display Mode

This function allows easy selection of a desired mode according to monitor application.

√: Calibration is available.

Mode	Purpose	
1-DICOM	Available to display in DICOM mode.	√
2-Custom	Available for the setting according to your preference.	√
3-CAL1/4-CAL2/5-CAL3	Displays the screen adjusted by calibration software.	√
6-Hybrid-γ	Automatically identifies the display area for the monochrome and color images on the same screen, and displays each set status.	-
7-sRGB	Suitable for color matching with sRGB compatible peripherals.	-
8-Text	Suitable for displaying texts for word processing or spreadsheets.	-
9-ALT ^{*1}	Suitable for displaying the color with the brightness and grayscale appropriate for the environmental illumination.	-

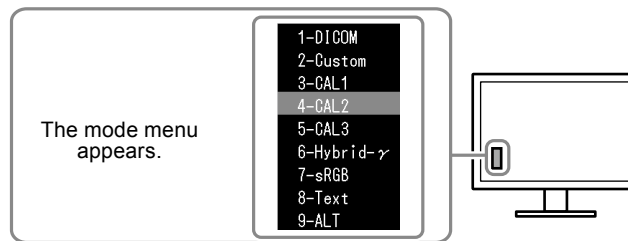*1 The items described above may not be displayed correctly depending on the usage environment. The quality of the monitor cannot be managed using this mode.


Procedure

1. Press MODE.

The mode menu appears at the bottom left of the screen.

2. The mode among the list is highlighted in turn each time MODE is pressed.

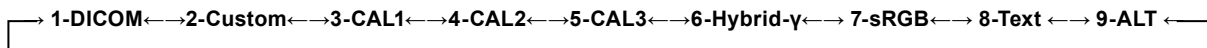You can switch the mode with ▲ or ▼ while the mode menu is displayed.

Controlling Monitor Quality

The quality control software “RadiCS LE” (for Windows) is included with this monitor.

To calibrate the monitor and manage the calibration history, install RadiCS LE with reference to the RadiCS LE User's Manual (on the CD-ROM). Use the RadiCS SelfQC function and built-in Integrated Front Sensor to perform periodic calibration and Grayscale Check for the monitor independently. For details, refer to the User's Manual (on the CD-ROM).

Attention

- Connect the monitor to the PC with a USB cable when using RadiCS LE.

Note

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit “RadiCS UX1”.

No picture


If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

Problem	Possible cause and remedy
<p>1. No picture</p> <ul style="list-style-type: none"> Power indicator does not light. <hr/> <ul style="list-style-type: none"> Power indicator is lighting green. <hr/> <ul style="list-style-type: none"> Power indicator is lighting orange. <hr/> <ul style="list-style-type: none"> Power indicator flashes orange and green. 	<ul style="list-style-type: none"> Check whether the power cord is connected properly. Turn the main power switch on. Press . Turn off the main power, and then turn it on again a few minutes later. Increase "Brightness", "Contrast", or "Gain" in the Adjustment menu. Turn off the main power, and then turn it on again. Switch the input signal. For details, refer to the User's Manual (on the CD-ROM). Operate the mouse or keyboard. Check whether the PC is turned on. When Presence Sensor is set to "Auto" or "Manual", the monitor may have shifted to the power saving mode. Come near to the monitor. Turn off the main power, and then turn it on again. There is a problem in the device that uses DisplayPort connection. Solve the problem, turn off the monitor, and then turn it on again. Refer to the user's manual of the device that uses DisplayPort connection for details.
<p>2. The message below appears.</p> <ul style="list-style-type: none"> This message appears when no signal is input. Example: <div data-bbox="316 1077 627 1274" data-label="Image"> </div> <ul style="list-style-type: none"> The message shows that the input signal is out of the specified frequency range. Example: <div data-bbox="316 1391 627 1626" data-label="Image"> </div> <p>fD: Dot Clock fH: Horizontal Scan Frequency fV: Vertical scan frequency</p>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <ul style="list-style-type: none"> The message shown left may appear, because some PCs do not output the signal soon after power-on. Check whether the PC is turned on. Check whether the signal cable is connected properly. Switch the input signal. For details, refer to the User's Manual (on the CD-ROM). Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions"). Reboot the PC. Select an appropriate display mode using the graphics board's utility. Refer to the manual of the graphics board for details.

RadiForce® RX440

LCD-Farbmonitor

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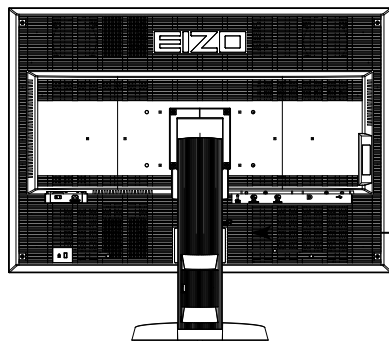WARNING
RISK OF ELECTRIC SHOCK. DO NOT OPEN.

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